



Hospital 2 Home Packs



NephSys H2H Packs

Product code	Product content
H2H 1720S 	2 x nephrostomy bags NS721.1720S 2 x night bags NB2 1 x NSbelt.02
H2H 1730S 	2 x nephrostomy bags NS721.1730S 2 x night bags NB2 1 x NSbelt.02
H2H 3720S 	2 x nephrostomy bags NS721.3720S 2 x night bags NB2 1 x NSbelt.02
H2H 3730S 	2 x nephrostomy bags NS721.3730S 2 x night bags NB2 1 x NSBelt.02



1 x belt
(includes two drainage bag suspenders)



All NephSys H2H packs come with literature content.
See page 2.

Literature content

1.

MANFRED SAUER UK

The #1 solution for nephrostomy patients...



...NephSys – the nephrostomy bag and belt system

2.

The care and management of your nephrostomy system

by Carol Hutchings RGN, Urology Specialist Nurse

Sponsored by
MANFRED SAUER UK

3.

MANFRED SAUER UK

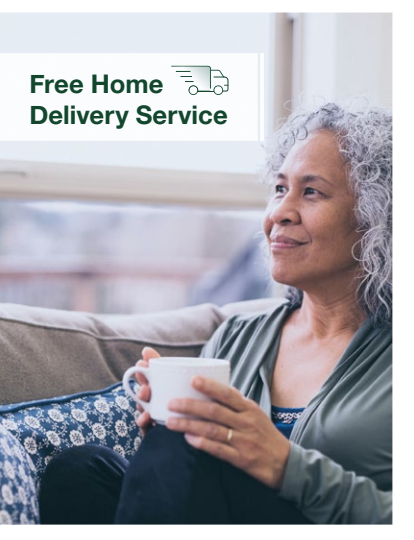
MY NEPHROSTOMY PASSPORT

Carry this passport with you to medical appointments and when travelling.

4.

MANFRED SAUER CARE

Free Home Delivery Service



5.

MANFRED SAUER UK

For help, advice and sample requests, please contact our helpline:

01604 595 696

helpline@manfred-sauer.co.uk

www.manfred-sauer.co.uk

MANFRED SAUER CARE
NHS Prescription Dispensing Services

To learn more about our FREE Home Delivery Service, please contact:

FREEPHONE 0800 999 5596

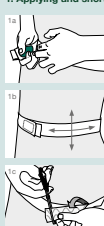
prescriptions@manfredsauercare.co.uk

www.manfredsauercare.co.uk

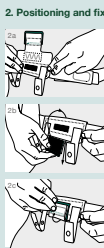
6.

NephSys – the nephrostomy bag and belt system

Instructions for use

- Applying and shortening the waist belt**


Place the side of the waist belt with the silicone lines on the skin (Fig. 1a) and put the belt on anywhere between the waist and the hips under slight tension (Fig. 1b). Ensure a comfortable but, most importantly, a stable fit.

The excess length of the belt can be cut off with scissors (Fig. 1c).
- Positioning and fixing the hook-and-loop pad**


Attach the hook-and-loop pad to the waist belt, ensure the position of the pad is in a convenient position for connecting to the catheter/emptying the nephrostomy system and is comfortable (tube isn't pulling) as in illustration (Fig. 2a), this would normally be between the level and hip or above the hip on the side where the drainage system is located.

After attaching the hook-and-loop pad (Fig. 2a) guide the green top tab around the belt (Fig. 2b) and then secure the hook-and-loop fixation strip (Fig. 2c).
- Attaching the bag to the hook-and-loop pad**


We recommend holding the inlet tube with one hand to prevent contact with the connecting area of the adapter. Attach the NephSys bag so that the outlet tap is facing forward. Thread the two retaining straps through the loops positioned at the top of the bag and fasten using the press studs (Fig. 3a).
- Connecting the bag to the catheter**


A luer lock connection consists of a male and female connector coming together and locking. Connect the male luer lock connector on the bags to the female luer lock connector on the nephrostomy catheter/tube, place together and turn clockwise to lock the connection (Fig. 4a).

Do not tighten the luer lock too much as this can cause the connection to crack.

Ensure that the guide strings are not caught in the thread when locking the two ends together, as this can cause leakage. Connect the adapter of the bag to the catheter without tension.
- Securing the inlet tube**


To reduce the risk of twisting or pulling on the inlet tube or putting strain on or dislodging the nephrostomy catheter/tube, the inlet tube is secured to the waist belt to prevent this from occurring.

Remove the protective films from the middle of the adhesive fixation tabs. Guide the inlet tube between the two adhesive surfaces and stick the two adhesive surfaces together with the inlet tube in the middle (Fig. 5a). Attach the tab to the belt where the inlet tube naturally crosses the waist belt using the hook-and-loop fixation strip (Fig. 5b).

For more information or samples, simply contact our helpline:

Tel: 01604 595 696

Email: helpline@manfred-sauer.co.uk

MANFRED SAUER UK

7.

**The #1 solution for nephrostomy patients...
...NephSys bag and belt system**

Available on prescription

NephSys – the nephrostomy bag and belt system consists of an elasticated, fully adjustable waist belt with hook and loop pads that can be worn on the belt in your chosen position. The drainage bags suspend from the waist belt and directly connects to the nephrostomy catheter. Cost-effective and available on FP10 prescription in the community.

- Secure and comfortable solution for nephrostomy patients from hospital to home.
- Designed in collaboration with patients and healthcare professionals.
- No more replacing nephrostomy tubes that have been pulled out or dislodged due to the traditional use of leg bag drainage systems.
- No more wasting valuable clinic time arranging for hospital products to be supplied to patients within the community.

Comfortable, undetectable under most clothes and connects to all universal right drainage bags.



MANFRED SAUER UK

For more information, please contact the helpline on:

01604 595 696

helpline@manfred-sauer.co.uk

www.manfred-sauer.co.uk

MANFRED SAUER CARE
NHS Prescription Dispensing Services

FREE home delivery service providing a personal service from a professional team.

For your FREE information pack contact 0800 999 5596 or email prescriptions@manfredsauercare.co.uk



Models used to safeguard patient privacy.

1. NephSys booklet
2. Care and management of your nephrostomy system booklet
3. My nephrostomy passport
4. Free Home delivery service booklet
5. Advice and sample card
6. NephSys instructions for use
7. NephSys leaflet

